

Technology and service for your hair consultations



The new platform for your hair consultations

Trichovision is the new FotoFinder solution for trichology. It will enhance your hair consultations in three dimensions: Firstly, it delivers brilliant trichoscopic and clinical images. Secondly, with just one mouse click you have access to diagnostic support through the exclusive TrichoLAB Service. And thirdly, the system offers comprehensive analysis functions for detailed hair diagnostics including the digital trichogram.

The precision optics for video trichoscopy

Leading hair professionals worldwide rely on the unmatched image quality of FotoFinder. It is also of great value in scientific studies. With the uninterrupted live zoom of the FotoFinder Full HD video camera and its precision lens, you can view the scalp structure, follicular units and individual hairs with a magnification of up to 120X and with an accuracy of half a micron. Taking a photo of a hair shaft with 70X magnification you will see incredible 148,000 pixels/mm². This precision is necessary for accurate diagnosis – and is made possible by the best optical magnification that only FotoFinder offers!




Setting the Pace in Skin Imaging Worldwide.

Test Clinic
Arbore-Schmid Str. 3
81544
Bad Reibach

TrichoLAB
Trichoscopy Report

Name: Tom	Surname: Test	Personal ID: 12345678901	Date: 17/03/2015
--------------	------------------	-----------------------------	---------------------

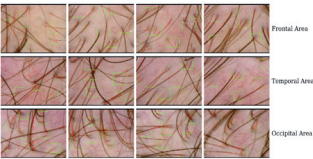

Statistics:

	Frontal Area	Temporal Area	Occipital Area
Average number of hairs (N/mm²)	203	140	197
Average hair shaft thickness (µm)	60	60	65
Thin (< 50 µm) hairs (%)	6	4	0
Short (< 50 µm) hairs (%)	26	15	0
Thick (> 50 µm) hairs (%)	69	81	92

Exclusively for FotoFinder customers: The TrichoLAB Service

Images can be transferred with a mouse click from Trichovision to the TrichoLAB service team where experts analyze your images and provide you with a detailed trichoscopic evaluation in form of a report, and this, without shaving, without depilation, and without biopsy! Thus, you can outsource the time-consuming measurement and evaluation processes of the various trichoscopic features. This saves time and gives you the basis for an accurate diagnosis.

Added value through analysis and digital trichogram

In order to quantify the exact dimension of hair loss as well as the treatment progress, the FotoFinder Trichoscale pro software supports you with a wide range of possibilities for hair measurement and analysis. It provides you with different parameters such as hair density, hair thickness, follicular units, anagen/telogen rate and more. The digital trichogram offers you a detailed hair root analysis and spares your patient painful depilation!




Cutting-edge technology and design for your practice

FotoFinder Trichovision combines brilliance and performance with elegant design. The result is a long-lasting and stylish all-in-one solution, which guarantees a superior visual experience. The FotoFinder universe software platform is intuitive to use and powerful in the network. The program allows an easy image management and provides detailed reports. Each system is tailored to your needs and can, in addition, be used for dermoscopy and mole mapping.



Your benefits at a glance:

- Unsurpassed quality in trichoscopy
- Unmatched accuracy of half a micron
- Uninterrupted optical zoom up to 120X
- Extensive measurement and analysis functions
- Calculation of hair density, thickness and follicular units
- Video trichogram with in depth reports
- Direct link to the TrichoLAB second opinion service
- Extensive support for your practice marketing
- Optional: integration of one or several iPads

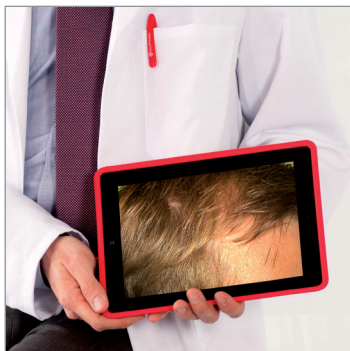

www.trichovision.com

iPad is a trademark of Apple Inc., registered in the U.S. and other countries.



09/2015